


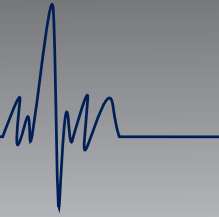


Solutions Start Here...



**Solutions Start Here**  
[www.smpcanada.com](http://www.smpcanada.com)

**SMP Canada**  
10 Sims Crescent, Richmond Hill  
Ontario, L4B-1K9 Canada  
Tel: (905) 764-7767 Fax: 1-866-627-9798  
[support@smpcanada.com](mailto:support@smpcanada.com)

## SMP 3000

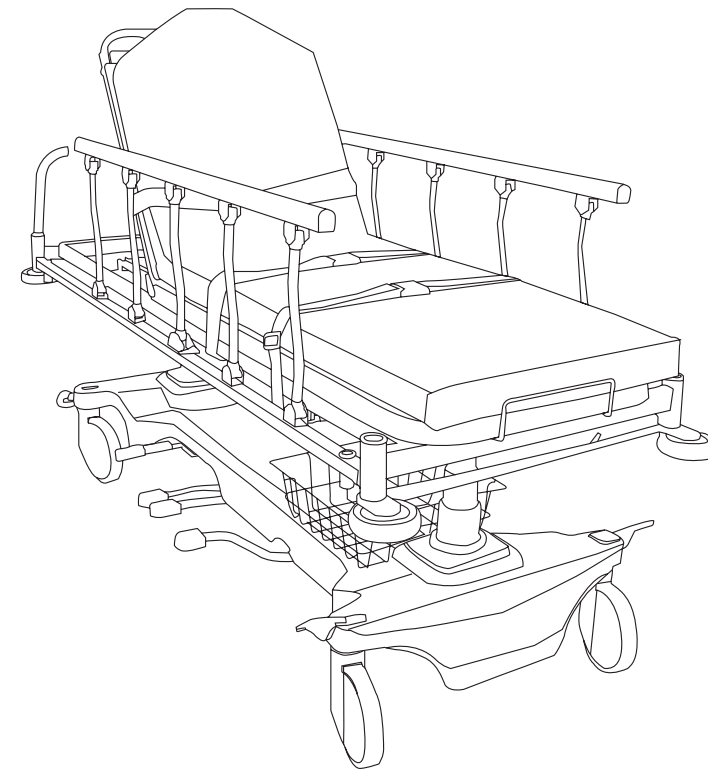

## USER & MAINTENANCE MANUAL




# USER MANUAL

## TRANSPORT STRETCHERS SMP 3000

### APPENDIX C SMP-3000

#### Hydraulic Emergency Stretcher – User & maintenance Manual

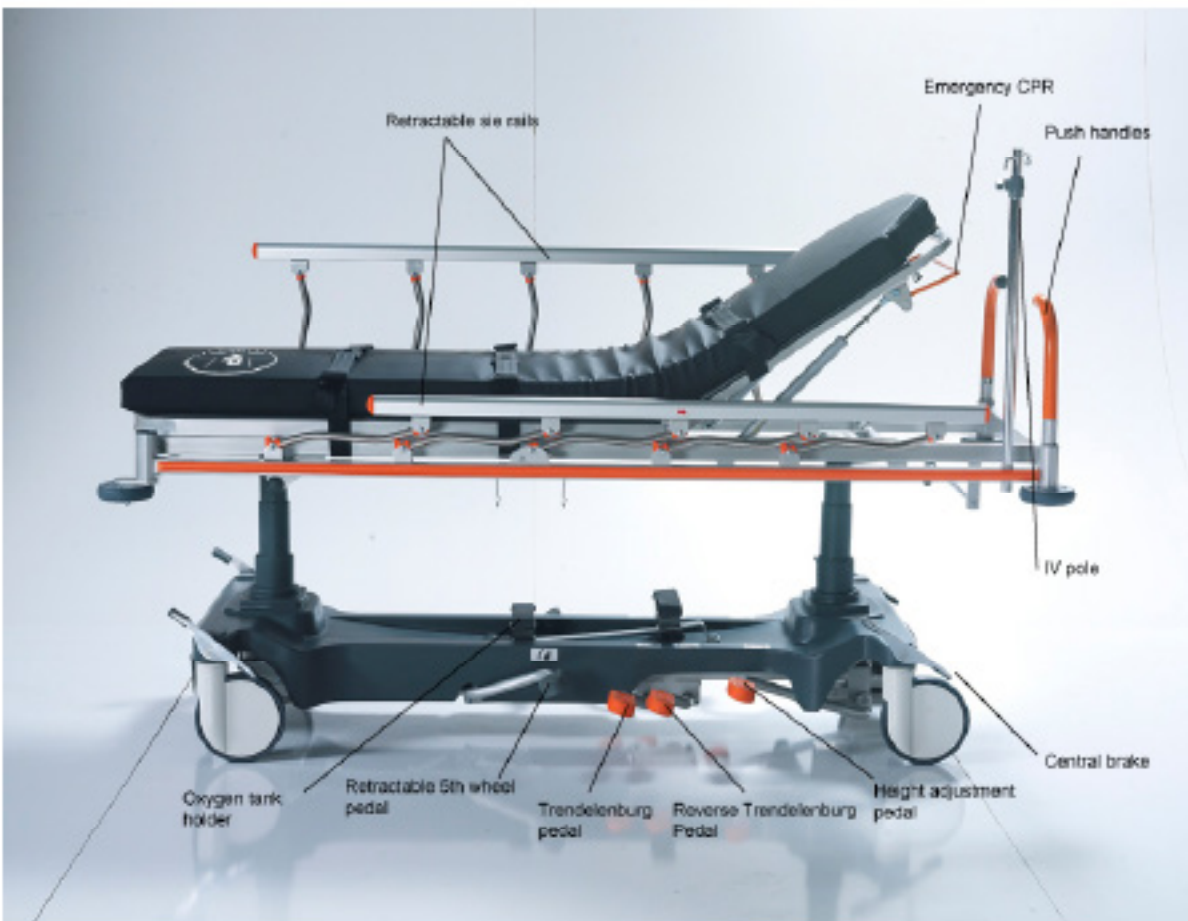



# USER MANUAL

## TRANSPORT STRETCHERS **SMP 3000**

### **General Remarks**

We congratulate you on your choice of a SMP-3000 hydraulic emergency stretcher and would like to thank you for your confidence in our company and its products. You have purchased a SMP-3000 hydraulic emergency stretcher, developed and manufactured in accordance with the current state of technical knowledge.

SMP Canada's stretchers fully meet all safety and operational requirements. They are tested in accordance with international standard and carry the CE seal. This confirms that they fulfill the safety requirements relating to medical products. This user manual is intended to give you and your staff practical information and tips on the safe and correct use of the SMP hydraulic emergency stretcher.

When putting the stretcher into service for the first time, or wanting to learn about its operation and maintenance, one must read and strictly obey this user manual. To avoid faulty handling and to ensure trouble-free operation of the stretcher, this documentation must be available to caregivers and/or patients at all times.

**Secondary Medical Products Canada Inc.**

**SMP Canada**

10 Sims Crescent

Richmond Hill, Ontario, L4B 1K9 Canada

Tel.: ++1 905-764-7767

Fax: ++1 866-627-9798

E-Mail: [info@smccanada.com](mailto:info@smccanada.com)

Website: [www.smccanada.com](http://www.smccanada.com)



# USER MANUAL

## TRANSPORT STRETCHERS SMP 3000

### Table of Contents

APPENDIX C.....	1
General Remarks.....	2
1. Safety Instructions.....	4
2. Stretcher Installation.....	4
3. Stretcher Usage.....	5
3.1 Movement Handles Usage.....	5
3.2 X-Ray Transmittance Specialty Usage.....	5
3.3 Side Rail Usage.....	5
3.4 Pedals.....	5
3.5 Stretcher Height Adjustment.....	6
3.6 Castor Lock Mechanism.....	6
3.7 Retractable 5 <sup>th</sup> Castor.....	6
3.8 IV Pole Usage.....	6
4. Stretcher Cleaning.....	6
5. Storage.....	7
6. Maintenance Inspection Schedule.....	7



# USER MANUAL

## TRANSPORT STRETCHERS SMP 3000

### 1. Safety Instructions

- Before using the stretcher's functions read these statements with detail and use this product in accordance with these statements. If necessary, place a copy of these instructions near the product.
- If you notice the product is not working, do not use it and contact SMP's technical service department.
- The safe working weight is 150 kg and do not exceed this. If there is an obligatory situation to exceed this limit, the surface which the patient lies on should be adjusted to lowest position and the functions of the stretcher should not be used.
- When the patient is on the stretcher, the castors should always be locked. (Except the movement of the stretcher to another place)
- When the patient is on the stretcher, a second person should not sit on the stretcher.
- The patient and the attendant are always in danger in the following situations.
  - In the course of taking the stretcher away from one place to another on crinkled or grungy surface
  - In the course of exceeding the safe working weight
  - In the course of placing the moving parts carelessly
- Only the approved brands' parts should be used. When the replacement parts which are provided from the other suppliers are used, the stretcher manufacturer company will not accept responsibility for any damage, loss or injury.

### 2. Stretcher Installation

During the installation of the stretcher, please obey the following rules:

- Open the package of the stretcher completely and carefully
- Check the stretcher for the possible damages during the carriage
- Check the accessories and the main parts of the stretcher for any deficiency carefully, in case of any deficiency, contact the service department
- Before taking action about the stretcher, read the user's manual carefully
- Install the accessories as specified in user's manual

#### **\*WARNING\***

**DO NOT PUT ANYTHING ON THE BOTTOM FRAME COVER**



# USER MANUAL

## TRANSPORT STRETCHERS SMP 3000

### 3. Stretcher Usage

#### 3.1 Movement Handles Usage

Pull the handles placing both sides of the headrest part of the stretcher upward to get into use. After ending your movement, put the handles into their holes and fold them down.


#### 3.2 X-Ray Transmittance Specialty Usage

X-Ray transmittance panel (placed on the upper frame of the bed) can be moved up and down on the panel system by using the small stick (near the transmittance panel). After adjusting to required place, this panel can be locked by using the locking system (placed on the stretcher).

#### 3.3 Side Rail Usage

To use the side rails, pull them (when they are in closed position) upward. When you hear a "click" sound, this means the side rails are locked. To put down the side rails, lift the side rail lock lever by hand and push the side rail downward at the same time.

#### 3.4 Pedals

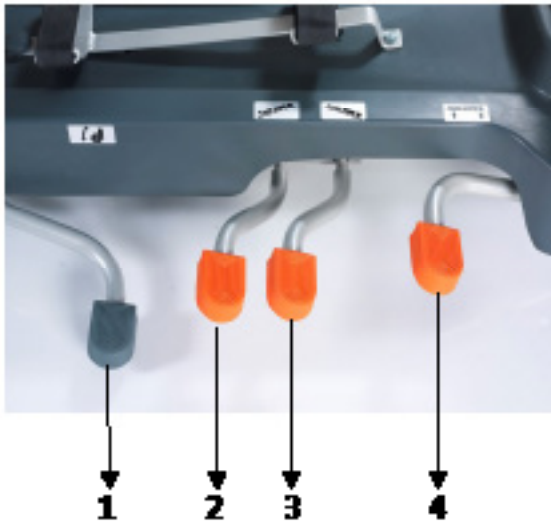

1	HEIGHT ADJUSTMENT
2	TRENDELENBURG
3	REVERSE TRENDELENBURG
4	FIFTH CASTOR



# USER MANUAL

## TRANSPORT STRETCHERS SMP 3000

### 3.6 Castor Lock Mechanism

When you push the pedal on the castor lightly and bring it into down position, all castors are locked and prevent the movement of the stretcher. When you lift the pedal and bring into vacancy, all castors are free. To bring the anti-static direction castor into use, push the pedal upward lightly.

### 3.7 Retractable 5<sup>th</sup> Castor

Fifth castor is used for more comfortable movement of the stretcher to required directions. To use the fifth castor push the fifth castor pedal downward and to retract the fifth castor, push the pedal upward.

### 3.8 IV Pole Usage

Place the IV Pole accessory into the hole (placed on the headrest of the stretcher) and fix it. To lengthen IV Pole, open the lock mechanism by turning it clockwise. Pull the IV Pole upward. To fix in the required position, turn the lock system counter clockwise.

## 4. Stretcher Cleaning

The stretchers should be cleaned with a wet duster or sponge. Please do the cleaning after discharging the patient.

### **\*WARNING\***

**NEVER USE ACID, DISSOLVENT, DETERGENT, PHOSPHATE OR ANY CHEMICAL INCLUDING PHOSPHATE OR ANY OTHER CORROIVE AGENTS.**



# USER MANUAL

## TRANSPORT STRETCHERS **SMP 3000**

### 5. Storage

For the long term storage of the stretcher, obey the following instructions:

- Be sure that all the wheels of the stretcher are on a level surface
- Do not place any other stretcher or a heavy subject on the bed

### 6. Maintenance Inspection Schedule

TECHNICIAN NAME	SIGNATURE	DATE	EXPLANATION





# USER MANUAL

## TRANSPORT STRETCHERS **SMP 3000**
